© Sensitest - HCG dipstick

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**Instructions for use Sensitest Pregnancy test dipstick (hCG)**

Carefully read the instructions for use before you start testing. As soon as you are pregnant, your body produces the pregnancy hormone Human Chorionic Gonadotrophin (hCG). The quantity of hCG doubles every 2 to 3 days and reaches a peak after some 7 to 12 weeks of pregnancy. This test detects the pregnancy hormone in your urine. When the strip is immersed into a urine specimen, capillary action carries the specimen to migrate along the membrane. When hCG in the sample reaches the test zone region of the membrane, it will form a colored line. Absence of this colored line suggests a negative result. The Sensitest pregnancy test dipstick can already show the presence of hCG before you have your period. However, to obtain a reliable result, we recommend carrying out the test from the day that you expect to menstruate. If you have an irregular cycle, you must use the longest cycle you have had during the past six months as a criterion.

**Timing:** The test can be carried out at any time of day.

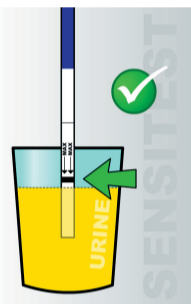
**What you need before you start testing**

1. A sealed foil package with a Sensitest pregnancy test dipstick.  
Do not use a pregnancy test from an opened or punctured pack.
2. A clean, dry receptacle or plastic cup.
3. Urine from the woman whose pregnancy must be confirmed.

**The test in 7 steps**

1. Take a closed pack and check that it indicates "Pregnancy test".  
Do not use a test that has passed its shelf life.
2. Collect a little urine in a clean cup.
3. Open the pack with the test with dry, clean hands.
4. Hold the dipstick at the blue end and dip the other end into the urine for about ten seconds.
5. This part may not be dipped beyond the thick line near "MAX" (see image).
6. Place the test on a smooth, dry base.
7. Wait 5-10 minutes before reading the result. It is important that the background is no longer discolored.

However, do not wait more than 30 minutes before reading the result.

**Test result**

- The result is **negative** if a single line appears (image 1). This is the control line (C) to see whether you have carried out the test correctly. The test line (T) is not visible. You are not pregnant.
- The result is **positive** if two lines appear (image 2). You can clearly see a control line (C) and a test line (T). Even if the test line is faint, hCG has been detected in your urine. You are pregnant (image 3).

**Result: no lines.** If no lines are visible, the test has probably been carried out incorrectly. The dipstick has possibly absorbed too little urine. Hold the tip of the dipstick in the urine again for a few seconds.

**If the result is negative (not pregnant).** If the test indicates that you are not pregnant, the Sensitest pregnancy test dipstick has been unable to demonstrate the pregnancy hormone. You are not pregnant or you may possibly have carried out the test too early. You can carry out a new test after 3 days. If despite a negative outcome you do not menstruate within one week, we advise you to contact your family doctor or gynaecologist.

**If the result is positive (you are pregnant).** If the test indicates that you are pregnant, you should advise your family doctor in order to receive the correct attention during your pregnancy.

**Very light test result:** if the test line (T) is very light you could carry out a new pregnancy test after 48 hours.

**Restriction 1: Following pregnancy, miscarriage, sterility treatment**

It takes around 9 weeks after confinement or miscarriage before the hCG pregnancy hormone has disappeared entirely from your body. Up to that point the pregnancy test produces a positive result although you are not pregnant. Sterility treatments, based on hCG, may cause false results. Consult your physician.

**Restriction 2: Reliability of the pregnancy test**

With a 99% reliability, the Sensitest pregnancy test dipstick is just as reliable as a comparable test by your family doctor. Nonetheless, only your family doctor or gynaecologist can provide absolute certainty as to whether you are pregnant or not evaluating all clinical and laboratory findings. Your personal medical situation may influence the result of this test. Always consult your family doctor on pregnancy or if in doubt about pregnancy.

**Restriction 3: Do not use alcohol**

Alcohol may influence the test result. Do not use pregnancy tests after taking alcohol.

**Restriction 4: Elevated levels of hCG**

Elevated levels of hCG can be caused by a few conditions other than pregnancy. Therefore, the presence of hCG in urine specimen should not be used to diagnose pregnancy unless these conditions have been ruled out.

**Restriction 5: Applying the test**

This test may be used only with collected urine. If you require a pregnancy test to hold in the urine stream, use the Sensitest pregnancy test midstream.

**Restriction 6: Do not drink too much**

Drinking large amounts of water or other liquids like coffee or tea could dilute your urine and produce light or unclear test results. You are advised to refrain from drinking large quantities of liquids during your testing period. Preferably stop drinking 2 hours before testing. When a urine sample is too dilute, the test result could be negative. If pregnancy is still suspected, another urine specimen should be collected 48 hours later and tested again.


**Restriction 7: Interpretation of the test results**

A normal pregnancy cannot be distinguished from an ectopic pregnancy based on hCG levels alone. A spontaneous miscarriage or missed abortion could give at first a light positive and later negative test results.

**Restriction 8: Testing before your period starts**

The Sensitest pregnancy test can demonstrate hCG in urine before you expect your period. However, the test result is more reliable when tested on the day you expect your period or later. If you carry out a Sensitest pregnancy test before your period begins, you are advised to do another pregnancy test on the day you expect your period.

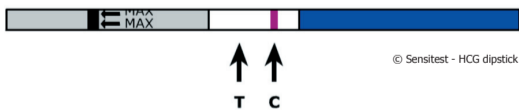*The text of these instructions was revised on 29 October 2013. Version 1.5*

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